

## ENSURE360 PENDANT

The Ensure360<sup>SM</sup> Pendant, by Sentric, is the mobile transmitter that provides peace of mind by allowing residents to place emergency calls from anywhere in the community. By communicating with the Locator infrastructure, the Pendant can provide useful location data to show caregivers where residents are with room-level accuracy.

Worn as a lanyard or on a watch band, the Pendant will always be with the resident to ensure staff can locate them as needed. The Pendant is sealed, water-resistant, and safe to use in the shower.

When a resident pushes the button, it sends a wireless signal to the Extender network to create an emergency call. It then sends information out rapidly to allow the Ensure360 platform to determine its location.

### Installation Instructions

#### Band

1. Gather the Pendant and Band to change the band/lanyard on
2. Flip the Pendant over and find the pin connecting the Lanyard clip to the Touch
3. Using the provided tool or a small flat-head screwdriver, carefully insert the tool between the Lanyard clip and the Pendant and pull the pin toward you, compressing it
4. Slowly and carefully lift the compressed watch pin out from the mounting hole



#### Specifications

**Dimensions:** 1.55" L x 1.55" W x 0.6" H

## **ENSURE360 PENDANT**

5. Remove the pin from the Lanyard clip and insert it into one end of the band
6. Depending on the desired orientation of the Pendant, connect the Band by inserting one end of the pin into one of the pin holes on the Pendant
7. Using the tool from earlier, compress the pin and line up the opposite end with the pin hole, release compression slowly

### Lanyard

1. Gather the Pendant and Band to change the band/lanyard on
2. Follow the steps above to remove the Band from both ends of the Pendant
3. Depending on the desired orientation of the Pendant , connect the Lanyard clip by inserting one end of the pin into one of the pin holes on the Pendant
4. Compress and line up the other end of the pin, attach lanyard to clip when finished

### Parts

- Pendant.Silver
- Pendant.Carbon
- Band.Charcoal
- Band.Navy
- Band.Lavender
- 693403 (Lanyard 10 pack)
- 693409 (Lanyard clip 10 pack)

### Technical Specifications

#### LED Patterns:

- Pendant transmitting an alarm - Blue flash
- Pendant transmitting location data - Green flash
- Pendant being updated - Green and yellow flash

Supervision interval - 15 minutes

Estimated battery life - Two years or longer

## **ENSURE360 PENDANT**

### **THE ENSURE360 SOLUTION**

Ensure360 is the perfect combination of Real-Time Location System (RTLS), environmental sensors, wearables, motion detectors, and traditional emergency call equipment designed to create a bubble of protection for your residents. The system, comprised of: a Headend server, a Desk console, a wireless mesh network made up of Extender access points, the Director network coordinator, and an array of Locator location beacons, will communicate with various end devices around the community to create a modular system to fit your needs! The end devices for Ensure360 include: the Pendant for tracking resident and staff location and providing a way to call for help from anywhere, the corded Pull Station, the touch pad Push Station, the Universal Transmitter with 2 dry contacts, and the Motion Sensor. Ensure360 will provide you with the most powerful emergency call system you've ever used.